⚠️ W A R N I N G S

WARNING: CAREFULLY READ FITTING INSTRUCTIONS AND WARNINGS PRIOR TO USE. TO ENSURE PROPER PERFORMANCE OF THE BRACE, FOLLOW ALL INSTRUCTIONS.

WARNING: IF YOU EXPERIENCE, OR HAVE AN INCREASE IN PAIN, SWELLING, SKIN IRRITATION, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.

WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM. CONSULT YOUR LICENSED HEALTH CARE PROFESSIONAL REGARDING SAFE AND APPROPRIATE ACTIVITY LEVEL WHILE WEARING THIS DEVICE.

CAUTION: FOLLOW APPLICATION AND CARE PROCESS DESCRIBED IN INSTRUCTIONS.

CAUTION: FOR SINGLE PATIENT USE ONLY.

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

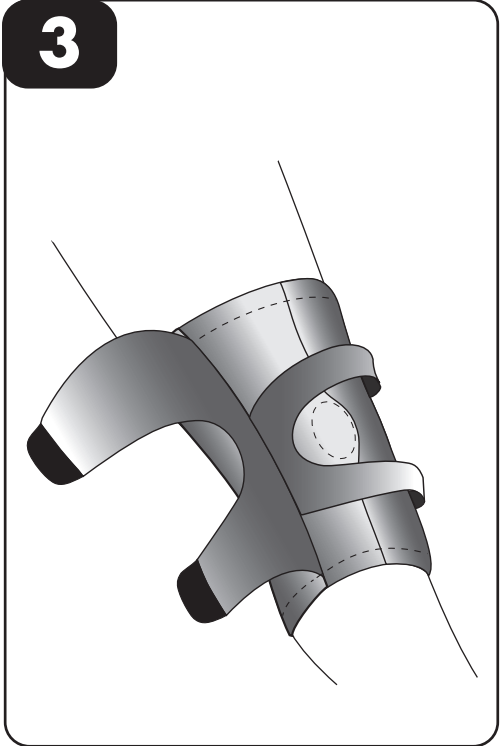
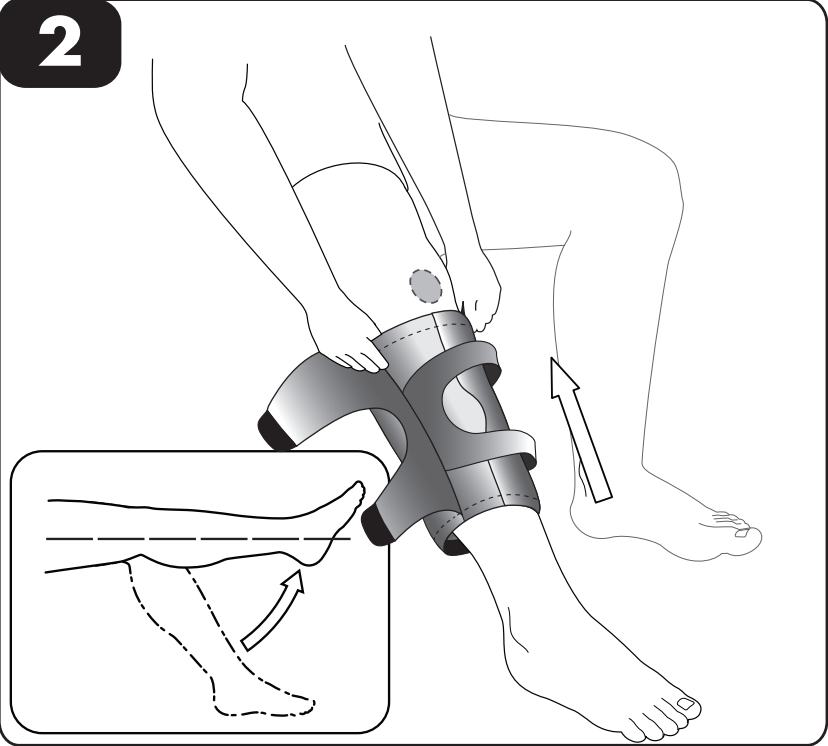
4

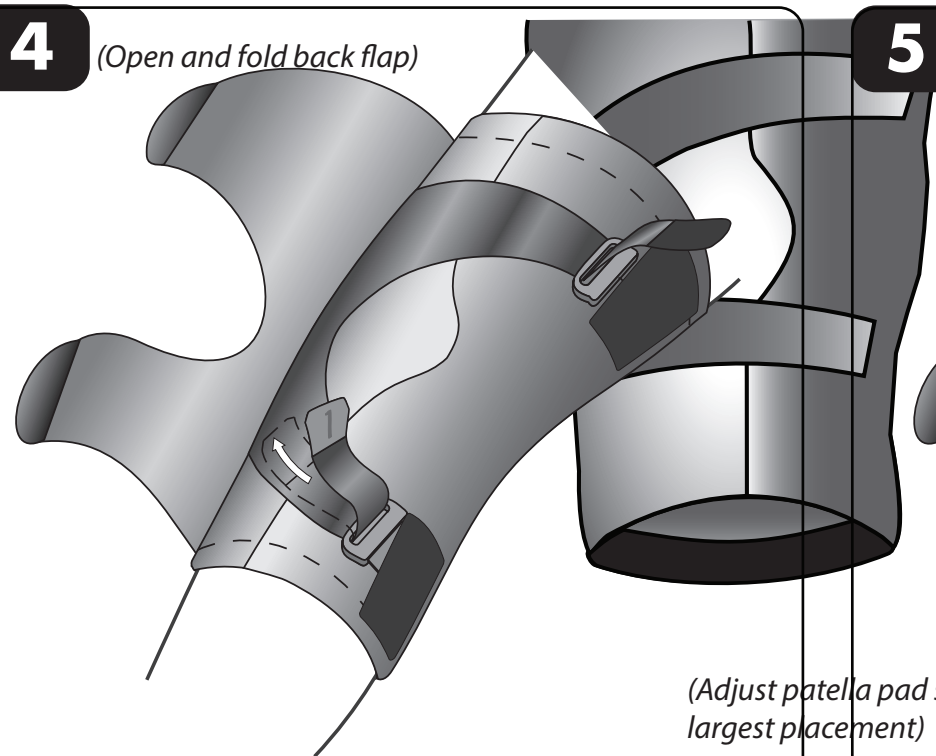(Open and fold back flap)


(Adjust patella pad straps to the largest placement)

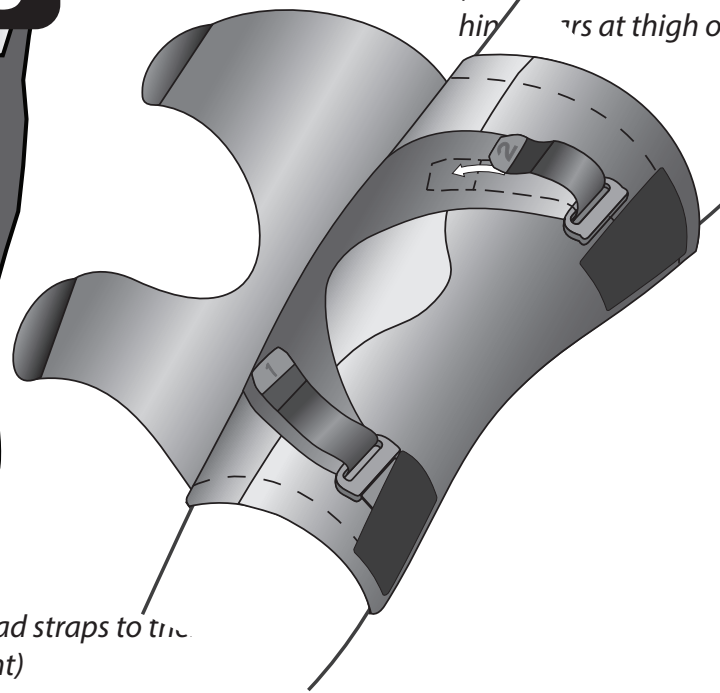5

(Grasp brace on to of hip
straps at thigh opening)

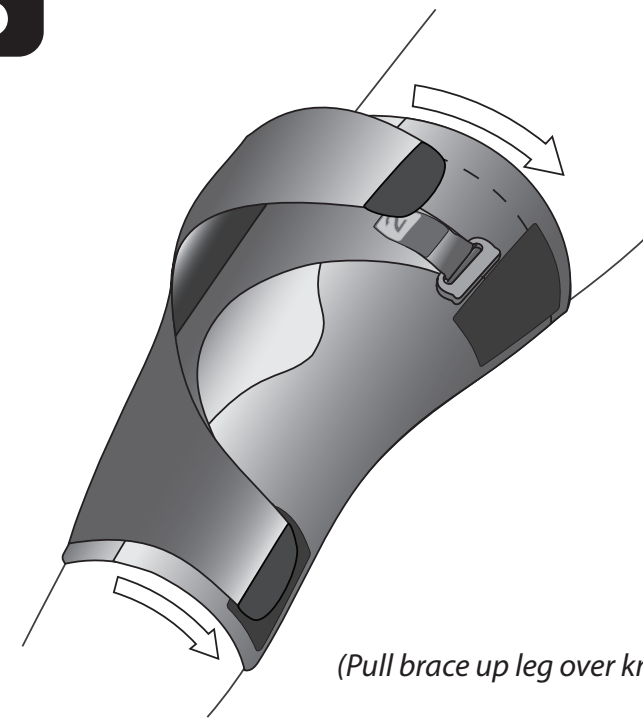6

(Pull brace up leg over knee)

(Center brace on leg using
patella guide marker)

- TIPS FOR USE AND CARE**
- Hand wash in cold water with a mild detergent, rinse and lay flat to air dry.
 - Do not machine wash or tumble dry.
 - After exposure to salt water or dirt, thoroughly rinse the brace with fresh water and allow to air dry.

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(Keeping leg fully extended, tension
the upper patella pad strap (2))

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(With leg fully extended tension lower
patella pad strap (1))

(Close flap over knee and
secure hook fasteners to landing
areas on medial side of leg)

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©2014 Breg, Inc.